DE EN FR IT

**Produktinformationen** / Product information  
Information sur le produit / Informazioni sul prodotto

# siegmund®

## FFP2 Atemschutzmaske / Particle filtering half mask Masque filtrant jetable / Facciale filtrante monouso

**CE 2534**  
EN 149:2001+A1:2009 FFP2 NR

<b>Produktname</b>	<b>FFP2 Atemschutzmaske</b>
Product name / Type de produit / Tipo di prodotto	FFP2 Particle filtering half mask / Masque filtrant jetable FFP2 / Facciale filtrante monouso FFP2
<b>Modell</b>	<b>GD7020</b>
Model / Modèle / Modello	
<b>Verordnung</b>	<b>(EU) 2016/425</b>
Regulation / Réglementation / Regolamento	
<b>Material</b>	<b>50% Polyester + 50% Meltblown</b>
Material / Matériel / Materiali	50% Polyester + 50% Melt-blown / 50% Polyester + 50% Meltblown / 50% Poliestere + 50% Meltblown
<b>Gültigkeitszeitraum</b>	<b>4 Jahre nach Produktionsdatum, siehe innenliegendes Zertifikat</b>
Validity period / Durée de validité / Durata della validità	4 years after production date, see certificate inside / 4 ans après la date de production, voir certificat à l'intérieur / 4 anni dalla data di produzione, vedere il certificato all'interno
<b>Artikelnummer</b>	<b>3400</b>
Article number / Numéro de l'article / Numero dell'articolo	
<b>Prüfnorm</b>	<b>EN 149:2001+A1:2009 FFP2 NR</b>
Test standard / Norme de contrôle / Standard di prova	
<b>PPE Ausstellende Behörde</b>	<b>Institut Technologii Tekstylnych CERTEX Sp. z o.o</b>
PPE issuing authority / Autorité de délivrance DPI / Autorità emittente DPI	
<b>Adresse der ausstellenden Behörde</b>	<b>ul. Gornicza 30/36, 91-765 Lodz - Poland</b>
Address of issuing authority / Adresse de l'autorité émettrice / Indirizzo dell'autorità emittente	
<b>Notifizierte Stelle</b>	<b>NB2534</b>
Notified Body / Organisme d'inspection notifié / Organismo di controllo notificato	
<b>Import in die EU &amp; Vertrieb</b>	<b>Siegmund Care GmbH</b>
Import to the EU & Sales / Importateur et distributeur dans l'UE / Importatore & Distributore nelle EU	Landsberger Straße 180, 86507 Oberottmarshausen, Deutschland / Germany / Allemagne / Germania, Telefon: (+49) 0 82 33 / 84 89 19 - 0, Internet: www.siegmund.care
<b>Hersteller</b>	<b>Automax Industrial Limited</b>
Manufacturer / Fabricant / Fabricante	No.75, Baowang Road, Gaoqiao Ind zone, Ningbo, 315174, China

DE EN FR IT

## Produktinformationen / Product information Information sur le produit / Informazioni sul prodotto

# siegmund®

### DE Warnhinweise

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### EN Warnings

1. The respirator is disposable, please change within 8 hours.
2. Make sure you wear the respirator properly and rightly.
3. Dispose properly after usage.
4. Please stop using the product if there is any discomfort.
5. Please ensure the mask is fitting tightly before entering contaminated area.
6. Please store out of childrens reach.
7. Avoid direct sunlight, solvents and frost condition during storage and transportation.
8. Please follow relevant health guideline or regulations while using the product.
9. Please read instruction carefully before using the product.

It can offer protection from firm and fluid deleterious kinds of dust, smoke, and aerosols. The particles may be fibrogenic – which means they irritate the respiratory system in the short term and can result in reduction of elasticity of pulmonary tissue in the long run. The total leakage may amount to a maximum of 11%. OEL transgression to the tenfold value.

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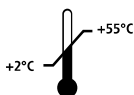
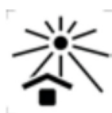
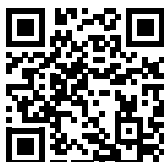
### EN Instruction Manual

1. Cup the respirator in your hand with the nosepiece at your fingertips allowing the headbands to hang freely below your hand.
2. Position the respirator under your chin with the nosepiece up.
3. Pull the top strap over your head resting it high at the top back of your head. Pull the bottom strap over your head and position it around the neck below the ears.
4. Place the fingertips of both hands at the top of the metal nosepiece. Mold the nosepiece to the shape of your nose by pushing inward while moving your fingertips down to both sides of the nosepiece. Pinching the nosepiece using one hand may result in less effective respirator performance.
5. The seal of the respirator on the face should be fit-checked prior to wearing in the work area.
  - a) Cover the front of the respirator with both hands. Being careful not to disturb the position of respirator.
  - b) Inhale sharply. A negative pressure should be felt inside the respirator. If any leakage is detected, adjust position of respirator and/or tension of strap. Retest the seal. Repeat the procedure until the respirator is sealed properly.

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